Description of this Thermometer

- ON/OFF button
- Display
- Battery compartment cover
- Measuring sensor / measuring tip
- Cleaning and disinfecting area (thermometer probe only)

This Digital Antimicrobial Medical Thermometer provides highly accurate readings over the human body temperature range. At the same time, it tests the microbe modal dispersion of microorganisms, the dispersion of contagious microorganisms, providing high safety to the user.

Antimicrobial Copper Properties

Surfaces made or covered by special copper alloys, have strong antimicrobial properties against a wide variety of microorganisms. Copper alloy emit antimicrobial copper ions Cu+ that whilst in contact with microbes and bacteria rupture their cellular membranes, thus destroying these microorganisms. This activity reduces the microbial flora on the coated area and – due to the halo phenomena – simultaneously causes a drastic reduction in pathogens on the remaining body of the thermometer. Thermometers containing parts with antimicrobial copper alloy drastically reduce microbial flora, minimizing the dispersion of contagious microorganisms and thus providing high safety to the end user.
**Efstathiou A. Panos "The Role of Antimicrobial Copper Surfaces in Reducing Healthcare associated Infections, European Infectious Disease, Volume 5, Issue 2, Autumn 2011.*

Important Safety Instructions

- Follow instructions for use. This document provides important production operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.
- This device is only to be used for measuring human body temperature through oral, rectal or axillary. Do not attempt to take measurements at other sites, such as in the ear, as it may result in false readings and may lead to injury.
- Do not use this device if you think it is damaged or notice anything unusual.
- We recommend cleaning this device according to the cleaning instructions before first use for personal hygiene.
- The minimum measurement time until the beep is heard must be maintained without exception!
- Consider that different measurement locations may require continued measuring even after the beep, see section «Measuring methods / Normal body temperature».
- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.
- Do not use this device close to mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices when using this device.
- Protect the device from impact and dropping!
- Avoid bending the thermometer probe more than 45°!
- Avoid ambient temperatures above 60 °C. NEVER hold this device! Use only the commercial disinfectants listed in the section «Cleaning and Disinfecting» to clean the device to avoid damage to the device.
- We recommend this device is tested for accuracy every two years or after mechanical impact (e.g. being dropped). Please contact Micro-life-Service to arrange the test.

WARNING: Do not rely on the measurement result given by this device is not a diagnosis! Do not measure the result on the measurement result only. Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

- Read the instructions carefully before using this device.

- Type BF applied part

Turning on the Thermometer

Turn on the thermometer, press the ON/OFF button (1), a short beep signals the thermometer ON. A display test is performed. All segments should be displayed.

The last measurement reading will be shown on the display (2) automatically for 2 seconds with the «M» icon.

The ambient temperature of less than 32 °C, an «L» and a flashing «C» appear at the display field (2). The thermometer is now ready for use.

Correct Function

Correct functioning of the thermometer is tested automatically each time it is turned on. If a malfunction is detected (measurement inaccuracy), this is indicated by «ERR» on the display, and a measurement cannot be possible. In this case, the thermometer must be replaced.

Using the Thermometer

Before use, keep this thermometer out of any physical contact, for at least 2 hours. This ensures the antimicrobial copper properties, as described in chapter «Antimicrobial Copper Properties». The use of antimicrobial copper is a supplement to and not a substitute for standard infection control practices. Users must continue to follow all current infection control and cleaning practices. We recommend cleaning the thermometer as described in chapter «Cleaning and Disinfecting».

Choose the preferred measuring method. When taking a measurement, the current temperature is continuously displayed and the «C» symbol and the battery level indicator are shown. Once you stop flashing, this means that the measured increase in temperature is less than 0.1 °C in 16 seconds.

To protect the battery life, turn off the thermometer by briefly pressing sensor (3). Otherwise, the thermometer will automatically turn off after about 10 minutes.

Storage of Measured Values

If the ON/OFF button (1) is pressed for more than 3 seconds when turning on the thermometer, the automatically stored maximum temperature during the last measurement will be displayed. At the same time, a «M» for memory will appear on the display. About 3 seconds after the battery is released, the maximum value disappears and the thermometer is ready for measurement.

Measuring methods / Normal body temperature

► In the armpit (axillary) / 34.7 - 37.3 °C
Wipe the underarm with a dry towel. Place the measuring sensor (4) into the arm into the center of the armpit so the tip is touching the skin and position the patient's arm next to the patient's body. This ensures that the room air does not affect the reading. Because the axillary takes more time to reach its stable temperature wait at least **5 minutes**, regardless of the body position.

► In the mouth (oral) / 35.5 - 37.5 °C
Do not eat or drink anything for 10 minutes before the measurement. The mouth should remain closed up to 2 minutes before starting a reading.

Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor (3) must be in good contact with the tissue. Close your mouth and breathe evenly through the nose to prevent the measurement from being influenced by inhaled/exhaled air.

Approx. measuring time: 10 seconds!

► In the anus (rectal) / 36.6 - 38.0 °C
Carefully insert the measuring sensor (4) of the thermometer 2 to 3 cm into the anal aperture.

The use of a probe cover and the use of a lubricant is recommended. If you are unsure of this measurement method, you should consult a professional for guidance/training.

Approx. measuring time: 10 seconds!

Cleaning and Disinfecting

For disinfection in home use environment, use a 70% Isopropyl alcohol swab, or a cotton tissue moistened with 70% Isopropyl alcohol to wipe surface pollutants off the thermometer probe (note: consider the application and safety instruction of the disinfectant manufacturer). Always start wiping from the end of the thermometer probe (approx. 4 cm) before wiping the rest of the thermometer tip. Afterwards the entire thermometer probe (see number (5) in the drawing) should be immersed in 70% Isopropyl alcohol for at least 5 minutes (max. 24 hours). After immersion, let the disinfectant dry off for 1 minute before use. Avoid immersing or wiping the display to protect it from fading.

The thermometer is not intended for professional use.

- MicroLife UAB

- P. Lukšio g. 32

- 08222 Vilnius

- Lithuania

- MicroLife Corporation

- 9F, 431, RuiGuang Road, Neihu

- Taipei 11492, Taiwan, China

Battery Replacement

When the «**W**» symbol (upside-down triangle) appears in the display, the battery is flat and needs replacing. To replace the battery remove the battery compartment cover (3) from the thermometer. Insert the new battery with the «+» at the top. Make sure you have a battery of the same type to hand. Batteries can be purchased at any electrical store.

Technical Specifications

Type:	Maximum thermometer
Measurement range:	Temp. < 32.0 °C; display «L» for low (too low) <p>Temp. > 42.9 °C; display «H» for high (too high)</p>
Measurement accuracy:	± 0.1 °C; 34 °C - 42 °C <p>± 0.2 °C; 32.0 - 33.9 °C and 41 - 42.9 °C</p>
Operating conditions:	10 - 40 °C; 15-95 % relative maximum humidity
Storage conditions:	-25 - +60 °C; 15-95 % relative maximum humidity
Battery:	LR41 (1.5V) /SR41 (1.55V)
Battery lifetime:	approx. 2700 measurements (using a new battery)
IP Class:	IP22
Reference to standards:	EN 12470-3, clinical thermometers; <p>ASTM E1112; IEC 60601-1; IEC 60601-1-2 (EMC); IEC 60601-1-11</p>
Expected service life:	5 years or 10000 measurements

This device complies with the requirements of the Medical Device Directive 93/42/EEC.

Guarantee

This device is covered by a **5 year guarantee** from the date of purchase. During this guarantee period, at our discretion, MicroLife will repair or replace the defective product free of charge.

Opening or altering the device invalidates the guarantee.

- The following items are excluded from the guarantee:
 - Transport costs and risks of transport.
 - Damage caused by incorrect application or non-compliance with the instructions for use.
 - Damage caused by leaking batteries.
 - Damage caused by accident or misuse.
 - Packaging checks and maintenance instructions for use.
 - Regular checks and maintenance (calibration).
 - Accessories and wearable parts: Battery
- Should guarantees service be required, please contact the dealer from where the product was purchased, or your local MicroLife service. You may contact your local MicroLife service through our website: www.microlife.com/support
- Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the original invoice.
- Repair or replacement without guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

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Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the original invoice.

Repair or replacement without guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

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The following items are excluded from the guarantee:

- Transport costs and risks of transport.
- Damage caused by incorrect application or non-compliance with the instructions for use.
- Damage caused by accident or misuse.
- Packaging checks and maintenance instructions for use.
- Regular checks and maintenance (calibration).
- Accessories and wearable parts: Battery

Should guarantees service be required, please contact the dealer from where the product was purchased, or your local MicroLife service. You may contact your local MicroLife service through our website: www.microlife.com/support

Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the original invoice.

Repair or replacement without guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

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The following items are excluded from the guarantee:

- Transport costs and risks of transport.
- Damage caused by incorrect application or non-compliance with the instructions for use.
- Damage caused by accident or misuse.
- Packaging checks and maintenance instructions for use.
- Regular checks and maintenance (calibration).
- Accessories and wearable parts: Battery

Should guarantees service be required, please contact the dealer from where the product was purchased, or your local MicroLife service. You may contact your local MicroLife service through our website: www.microlife.com/support

Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the original invoice.

Repair or replacement without guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

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The following items are excluded from the guarantee:

- Transport costs and risks of transport.
- Damage caused by incorrect application or non-compliance with the instructions for use.
- Damage caused by accident or misuse.
- Packaging checks and maintenance instructions for use.
- Regular checks and maintenance (calibration).
- Accessories and pieces of desgase: Battery

Should guarantees service be required, please contact the dealer from where the product was purchased, or your local MicroLife service. You may contact your local MicroLife service through our website: www.microlife.com/support

Compensation is limited to the value of the product. The guarantee will be granted if the complete product is returned with the original invoice.

Repair or replacement without guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

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The following items are excluded from the guarantee:

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